

ProFil Temp™

Temporary Cavity Filling Material



Prior to use, carefully read the instructions for use

ProFil™ Temp - Zinc oxide and synthetic materials temporary filling cement, self hardening, ready to use.
No expansion when setting, due to its high quality synthetic materials.
Eugenol-free to not hinder polymerization of resins and composites.
Adequate strength yet easily removed. Good adhesion to dentine and excellent marginal adaption.
Impermeable to medicaments, easy to use consistency.

Indications

Temporary filling of cavities
Temporary sealing of inlay preparations

Contraindications:

Profil Temp is contraindicated in patients with a history of allergic reaction to Zinc Sulfate or any other of the components.

Application

Remove the desired amount of cement from the jar, with a spatula. Tightly close the jar after use.
Using a suitable filling instrument, place material directly into cavity and form the contour. Hardening start immediately with saliva.
It is not essential to dry the cavity prior to placement. Temporary filling material hardens by absorption of moisture to give an impermeable seal. It is recommended that masticator pressure is not applied for approximately 15 minutes.
Remove it from the cavity with an ultrasonic device or with a probe.

Storage:

Store in a cool dry place (12°–25°C / 54°–77°F), away from humidity and intense light.
It is important that cap is tightly replaced after each use.

Shelf Life:

The lot number & expiry date are indicated on the product. Do not use after expiry date.

Caution:

Federal law restricts this device to sale by or on the order of a dentist. For Dental use only

Warning:

Profil Temp contains Zinc Sulfate which may seriously irritate eyes.
Avoid eye contact to prevent irritation and possible corneal damage. In case of contact with eyes, rinse immediately with plenty of water and seek for medical advice.
Harmful if swallowed. Don't give anything to drink. Do not induce vomiting and seek for medical advice.

Warranty:

Silmet Ltd. will replace product that is proven to be defective. Silmet Ltd. does not accept liability for any damage or loss, direct or consequential, arising from the use of or inability to use the product described. It is the responsibility of the dentist to determine before use, the suitability of the product for its intended use. The dentist assumes all risk and liability in connection therewith.



Silmet Ltd.

12 Hassadna st. Or-Yehuda 6022011, ISRAEL
Tel: 972-3-7353000 Fax: 972-3-5331581
info@silmet.co.il, www.silmetdental.com



Diadent

2 bis chemin du Loup, 93290 Tremblay en-France
Tel: 33-1-48012710 Fax: 33-1-42465230

SILMET