ProFil™ Temp

Temporary Cavity Filling Material (GB)

ProFil™ Temp temporary filling material is a ready to use temporary filling material with a synthetic resin base. Adequate strength yet easily removed. Good adhesion to dentine and excellent marginal adaption. Impermeable to medicaments, easy to use consistency.

INDICATIONS:

Temporary filling of cavities
Temporary sealing of inlay preparations
Temporary seal for medicaments
Temporary fixing of post crowns

APPLICATION:

Using a suitable filling instrument, place material directly into cavity and form the contour. It is not essential to dry the cavity prior to placement. Temporary filling material hardens by absorption of moisture to give an impermeable seal. It is recommended that masticatory pressure is not applied for approximately 15 minutes. For provisional fixing of post crowns spread a layer on inside of crown and seat in normal way.

NOTE:

For tube presentation, remove cap from tube, reverse and pierce membrane fully.

STORAGE:

Store in a cool dry place (5–25°C). It is important that caps are tightly replaced after each use.

SHELF LIFE:

3 years from date of manufacture

WARRANTY:

Silmet Ltd. will replace product that is proven to be defective. Silmet Ltd. does not accept liability for any damage or loss, direct or consequential, arising from the use of or inability to use the product described. It is the responsibility of the dentist to determine before use, the suitability of the product for its intended use. The dentist assumes all risk and liability in connection therewith.

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