

SECURAPPLIER™ PROGLASS CAPSULE APPLIER

Prior to use, carefully read the instructions for use

SecurApplier is a glass ionomer capsule applier, compatible for use with all encapsulated Glass Ionomer products. Use SecurApplier to pre-activate the capsule and dispense the mixed material from capsule directly to the designated surface. The applier is ergonomically designed to facilitate easy dispensing of the mixed materials.

Features

- Autoclavable, easy to clean
- No sharp edges - hand contoured design for maximum comfort
- Capsule lock enables controlled extrusion
- Stainless steel mechanical parts
- Robust & long lasting

DIRECTIONS FOR USE

Capsule Loading

After triturating, remove the capsule from the amalgamator: Insert and secure the capsule in capsule holder.


NOTE: The capsule cannot be inserted if the plunger is extended. Retract the plunger by pressing the release button.

Extruding the Material

Direct the capsule nozzle into the prepared surface of the tooth. Click trigger of applicator 3 times to prime the capsule. Click once more to extrude material. **No need for excess force.**

Removal of the Capsule

Press the release button to retract the plunger. Remove the used capsule from the applier by twisting clockwise and pulling upwards.

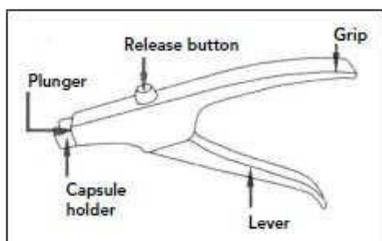

CLEANING

Clean the applier with a gauze soaked in alcohol.

STERILIZATION

SecurApplier can be autoclaved at 121-134°C (250-273°F) for 10 to 30 minutes (excluding heat-up time). Please follow manufacturer's instructions.

IMPORTANT: The use of chemical disinfectants/sterilants or ultrasonic cleaning may cause damage to the applier and are therefore not recommended.

SPECIAL NOTES

1. Prior to use, sterilize the applier.
2. Do not use the CAPSULE APPLIER on patients who have a history of allergy or show signs of allergy to metals. If any such reactions are experienced, discontinue the use of the product and refer to a physician.
3. An operator who has a history of allergy to metals should not handle the CAPSULE APPLIER. In case of allergy, discontinue use and consult a physician.
4. Take care not to get your hand or fingers caught between the lever and the grip.
5. This product should only be used by a dental professional.
6. This product should only be used for the applications described herein.

WARRANTY

Silmet Ltd. will replace product that is proven to be defective. Silmet Ltd. does not accept liability for any damage or loss, direct or consequential, arising from the use of or inability to use the product described. It is the responsibility of the dentist to determine before use, the suitability of the product for its intended use. The dentist assumes all risk and liability in connection therewith.



Silmet Ltd.

12 Hassadna st. Or-Yehuda 6022011, ISRAEL

Tel: 972-3-7353000 Fax: 972-3-5331581

info@silmet.co.il, www.silmetdental.com



Diadent

2 bis chemin du Loup, 93290 Tremblay en-France

Tel: 33-1-45913182 Fax: 33-1-48656723

 **SILMET**