"Strauss & Co." Dental Burs - General instructions for use

General instructions

For the safety of user and patient, it is highly recommended that only qualified dentists should work with dental diamond burs.

Attention should be paid to speed of work (RPM). In particular when working with a large head diameter and coarse or super coarse grit size. Using a bur with too high RPM level, could generate undesirable heat.

Do not force a bur into the turbine. In case of difficult access, check both turbine and bur. Remember! Strauss & Co. dental diamond burs are guaranteed and shall be replaced if found incompatible. However, special attention should be paid to instructions of use and maintenance of the turbine.

Strauss & Co.'s dental carbide & diamond burs are designed for several uses and are autoclavable.

Sterilize the burs before use.

Please note the FG mark on the package indicating the use in high speed turbine.

Max . RPM

ISO Ø (1/10 mm)	FG (RPM)	HP (RPM)
007-014	450,000	250,000
016-023	300,000	120,000
025-045	120,000	80,000
047-065	80,000	60,000
066-093	60,000	40,000

Use the bur with sufficient water coolant.

Apply a feather touch (up to 150 gram) while working with the bur.

Don't keep the bur still on one spot - Keep constant movement to avoid burning of the bur.

Please notice that there are separate instructions for Magic Touch and for pre-sterilized burs, they are available upon request.

Multi use burs - Sterilization instruction

Instructions for cleaning and sterilizing diamond dental burs

Disinfect and sterilize the burs before every use.

Cleaning:

Clean the burs by dipping them in disinfection solution (designed to use with dental burs) that contains a corrosion inhibitor for a minimum of 10 minutes. If needed - use brush to clean thoroughly. Rinse several times to remove all the disinfection solution.

Cleaning with Ultrasound:

Burs may be ultrasonically cleaned by inserting them in bur holders. A cycle of 5 minutes is recommended, using a general purpose cleaner. Rinse several times.

Sterilizing diamond burs – autoclave:

Place the burs in sterilization bags. Autoclave the burs for a minimum of 10 minutes at a temperature of 135°C.

Following sterilizing or cleaning methods, dry the burs and store in a clean and moisture free environment.

Note:

Use sterilizing and cleaning devices and materials according to the manufacturers recommended procedure. It is the responsibility of the user to ensure that sterilization is effective.

Tissue Trimmer

Use the Tissue Trimmer in a 45° on the targeted tissue.

Work with the trimmer in a slow and homogeneous pace on the targeted area in order to achieve accurate, smooth and coagulated result.

When additional coagulated is needed, apply the tissue trimmer on this specific spot.

Multi use burs – storage instruction

The burs should be stored in a bur stand or box until required. Storage should be in dry, clean conditions and at ambient temperature.

In such conditions – there is no expiry date for the burs.

Manufactured by STRAUSS & CO.

19 Hayezira St., Industrial Zone, Ra'anana 4336363, Israel. Tel: 972-9-7461631

Exclusive edition manufactured for SILMET Ltd. www.silmetdental.com

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Edizione esclusiva realizzata per SILMET Ltd. www.silmetdental.com
MedNet GmbH. Borkstrasse 10, 48163 Muenster, Germany.