# Septalloy NG 70, Securalloy

FR

CE 0473

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It is recommended to not directly expose the pulp to the amalgam.

## CAUTION : this product contains mercury

May be corrosive to metals. Fatal if inhaled. May cause damage to organs through prolonged or repeated exposure

## Pregnancy and breastfeeding:

Depositing of dental amalgam should be avoided in pregnant and breastfeeding women.  
The dental practitioner should evaluate, based on the individual clinical situation, the necessity of fitting dental amalgam over composite resin.  
The presence of dental amalgam in mothers does not contraindicate breastfeeding.

## Polymetalism:

The use of dental amalgams in direct or indirect contact with alloying elements from precious-metals, gold-plated anchorages (screw posts) or other metallic restorations should be avoided. Do not place dental amalgams close to other metal restorations, to prevent corrosion.

## Lichenoid lesions:

When local reactions occur, e.g. lichenoid lesions close to the amalgam, the restoration should be removed.

## Dental bleaching:

Performing bleaching (tooth whitening) on teeth containing amalgam is strongly discouraged.

## Practical recommendations:

- Do not place dental amalgams in direct contact with other types of metals.
- Dental amalgams are for single use only.
- Handle the product in well-ventilated premises with non-textile cleanable wall and floor coverings.
- Use this product with adequate ventilation (vapor exposure should be kept under 0.05 mg/m<sup>3</sup> or any lower limit set by any governmental or regulatory agency governing your use of the product).
- Wear appropriate safety equipment such as gloves, eyewear, masks and protective clothing when handling amalgam.
- The use of a rubber dam may decrease the amount of mercury absorbed by a patient during the removal or placement of an amalgam.
- Always use water, aspiration and a rubber dam when grinding and polishing the amalgam.
- Condense the amalgam using conventional methods (condenser). Do not use ultrasonic condensers. Use an enclosed amalgamator when mixing the self-activating capsules.
- Hygiene rules and good practice relating to the use of dental amalgam should be followed in order to limit as much as possible the concentration of mercury in the atmosphere of dental practices.
- Properly dispose of spent capsules and any excess unused amalgam.
- Use HgX, or similar type mercury-absorbing compounds in the event of spillage of the contents of the self-activating capsules.
- If a patient experiences a localized hypersensitivity reaction to dental amalgam, the amalgam should be removed. These precautionary procedures should always be used in addition to procedures recommended by your local regulatory agency and dental association.

## Effect of mercury on metals

Mercury corrodes and embrittles particular metals and their alloys. Avoid unnecessary contact between mercury and those metals (and their alloys).

## Spillages

Mercury presents a health hazard if handled incorrectly. Mercury is toxic by vapour inhalation and the effect is cumulative. Spillages of mercury should be removed immediately, including from places which are difficult to access. Use a plastic syringe to draw it up. Smaller quantities can be covered by sulfur powder and removed. Individual small droplets can be picked up by tin (i.e. Sn) foil and removed. Avoid inhalation of the vapour. During and after clean-up thoroughly ventilate the area where the spill has occurred. Vacuum cleaners should not be used.

## Moisture contamination

If moisture is introduced into the dental amalgam before it has set, properties such as strength and corrosion resistance can be affected adversely. If the alloy contains zinc such contamination can result in an excessive expansion (delayed expansion). Use a dry field, whenever it is possible.

## Waste disposal

Waste material and all primary containers that have held mercury shall be disposed of following appropriate management practice.

## Storage

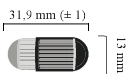
It is recommended that this product be stored below 25°C / 77°F in a well ventilated place. SDS available online at: [www.silmetdental.com](http://www.silmetdental.com)

Expiration Date: Do not use this product after its expiration date

## Presentations

Box containing 50 pre-filled capsules

Dimensions of a capsule

## SEPTALLOY NG 70

Spill 1	Spill 2	Spill 3
400 mg of powder	600 mg of powder	800 mg of powder
450 mg of mercury	690 mg of mercury	935 mg of mercury

## SECURALLOY

Spill 1	Spill 2	Spill 3
400 mg of powder	600 mg of powder	800 mg of powder
290 mg of mercury	435 mg of mercury	580 mg of mercury

## LIMITED WARRANTY – LIMITATION OF MANUFACTURERS' LIABILITY

The manufacturer's technical advice, whether verbal or in writing, is designed to assist dentists in using the product. Such advice does not expand the limited warranty or relieve the dentist of testing products to determine their suitability for the intended uses and procedures. The dentist assumes all risk and liability for damages arising out of the improper use of the product.

In the event of a defect in material or workmanship, manufacturer's liability is limited, at manufacturer's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to manufacturer. In no event shall manufacturer be liable for any indirect, incidental or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY MANUFACTURER, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

## SILMET Ltd.

12 Hassadna st., Or-Yehuda, 6022011, Israel  
Tel: 972-3-7353000 Fax: 972-3-5331581 info@silmet.co.il; www.silmetdental.com

## ECOREP

Diadent  
2 bis Chemin du Loup  
93290 Tremblay-en-France  
Tel: 33-1-45913182 Fax: 33-1-48656723

The products are exclusively manufactured for Septodont.

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EN

CE 0473

Prior to use, carefully read the instructions for use.

Non-gamma 2 Amalgam Alloys Self Activating Capsules  
Conforms to ISO 24234.

CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A DENTIST.

Keep out of the reach of children. For single use only. Do not place the device in direct contact with other types of metals.

### Indication for use

Intended for use as a dental restorative material in the treatment of dental caries. Dental amalgam is a material designed to restore permanent posterior teeth (molars and premolars) in cases of high prevalence of tooth decay and extensive and multiple lesions.

Dental amalgam can also be used when it is not possible to fit a dental dam, which is required to carry out adhesive treatment in the direct method, especially for extensive lesions.

### Contraindications

- Dental amalgams shall not be used:
- in patients with a known history of allergy to mercury, identified by a patch test.
  - in patients where the kidney has been weakened as a consequence of impaired kidney function.
  - for dental treatment of deciduous teeth, of children under fifteen years and of pregnant or breastfeeding women, except when deemed strictly necessary by the dental practitioner based on the specific medical needs of the patient.

### Identification of alloy content by capsule color

1 spill (400mg) Yellow cap  
2 spill (600mg) Green cap  
3 spill (800mg) Dark Blue cap  
The base color of the capsules identifies the device brand.

### Composition / Particle Shape (%)

	Ag	Sn	Cu	Zn	Hg	Spherical	Lathe Cut
Septalloy NG 70	69.5	19.5	10.5	0.5	53	20	80
Securalloy	56	28.8	15.2	None	40	100	-

All percentages are by weight.  
Powder weight may vary  $\pm$  3%  
The alloy to mercury ratio varies depending on the size and setting time.

### Triturating Time (seconds)

Amalgamator	Speed	1 spill	2 spill	3 spill
Silamat		4 - 5	5 - 6	7 - 8
Vari-mix II	H	10-15	15- 20	20- 25
Duomat (4000rpm)	3200	5 - 6	5 - 6	7 - 8
ProMix	High (Rabbit)	5 - 7	7 - 9	9 - 12

### SLOW SPEED AMALGAMATORS SHOULD NOT BE USED

Amalgams give satisfactory results with modern high speed vibrators.  
Not designed to be used with planetary mixers such as Copernicus and Capmix.

### Working Time (minutes)\*:

	Condense	Carving
Septalloy NG 70	2.5 - 4	4 - 5.5
Securalloy	2 - 3.5	3.5 - 5

\*working time may vary, depending on triturating conditions and technique applied  
Polish 24 hr. after placement.

### Mechanical properties

Compressive Strength (24hr): > 350 Mpa  
Dimensional Change:  $\leq$  0.15%  
Creep:  $\leq$  0.75%  
Note: The triturated amalgam should be bright and have a plastic consistency. If the amalgam is excessively plashy and wet looking and/or sets fast, decrease trituration time or amalgamator speed. A mix that appears excessively dry indicates that trituration time or amalgamator speed should be increased. Your clinical experience in obtaining a satisfactory working consistency will be the best guide to trituration times.

### Self Activating capsules – Instructions for Use

- Place and secure the capsule in the mixing arms of the desired amalgamator.
- To triturate, set the desired speed and time of the amalgamator.
- After trituration, remove the capsule from the amalgamator.
- Before opening, tap amalgam into the body of the capsule.
- With gloved hands, grasp the cap of the capsule with one forefinger and thumb, grasp the body of the capsule with the other forefinger and thumb, and pinch the cap while pulling apart. The cap should click apart.
- The amalgam in the body of the capsule should be tapped into an amalgam well.
- Inspect amalgam mass to ensure that the membrane from the mercury pillow pack is not mixed in with the amalgam. Remove the membrane from the amalgam if this is observed.
- Begin placement of amalgam.
- After placing the amalgam, snap the capsule body and cap back together and dispose of in accordance with all applicable regulations. Adherence to the ADA's current "Best Management Practices For Amalgam Waste (BMPs)" and "Dental Mercury Hygiene Recommendations" is strongly recommended.

### Placement of amalgam

- Insertion of the amalgam should commence immediately after trituration.
- Vertical condensing should begin immediately after trituration using instrument with flat smooth surface.
- Carving of the dental amalgam may begin immediately after completion of condensation.
- Relieve marginal ridge area with an explorer before removal of matrix band.
- A final polish will enhance the restoration's qualities. Conventional finishing and polishing techniques may be used.

### Special Note:

1. These alloys require less mercury than other alloys therefore, when triturating with dentomat adjust the regulating screw for a leaner mixture.
2. Spillages: "Mercury presents a health hazard if incorrectly handled. Spillages of mercury should be removed immediately according to local regulations."
3. Moisture contamination: "If moisture is introduced into the amalgam before it has set, properties such as strength and corrosion resistance may be affected adversely. If the alloy contains zinc, such contamination may result in an excessive expansion (delayed expansion). Whenever it is possible, use a dry field."
4. Refer to American Dental Association Publication, "Recommendations in Dental Mercury Hygiene".

### Warnings and precautions for use

Ensure to inform the patient (especially vulnerable persons such as the children and the older people) thoroughly about the choice of the filling material, and the known benefits and risks related to these restoration techniques.