




InLine Motion


Face Shield Assembly Instructions

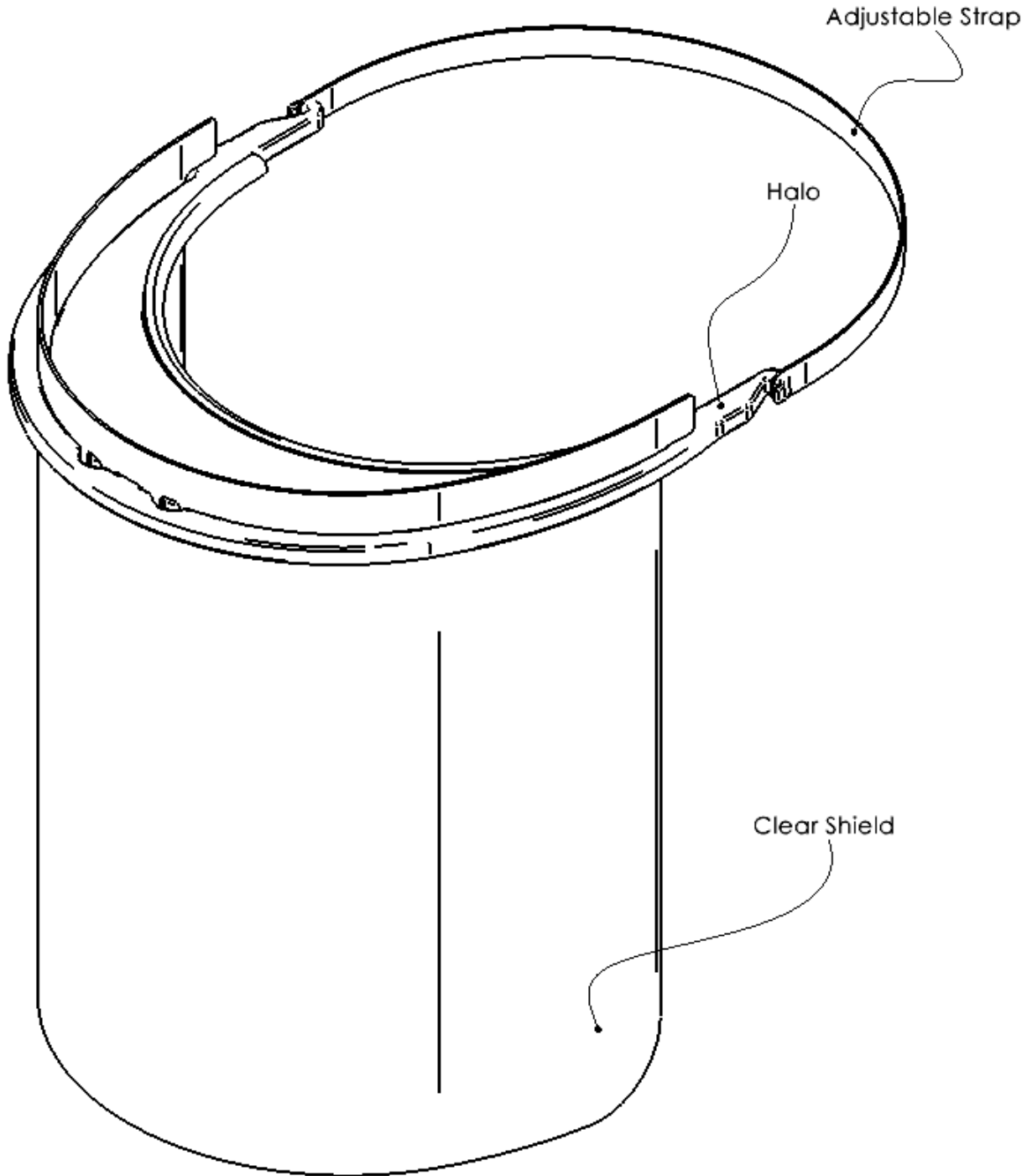


Step 1: Identify Halo and Clear Shield
Remove protective covering from **Clear Shield**



Step 2: Bend inner Halo to allow insertion of Clear Shield

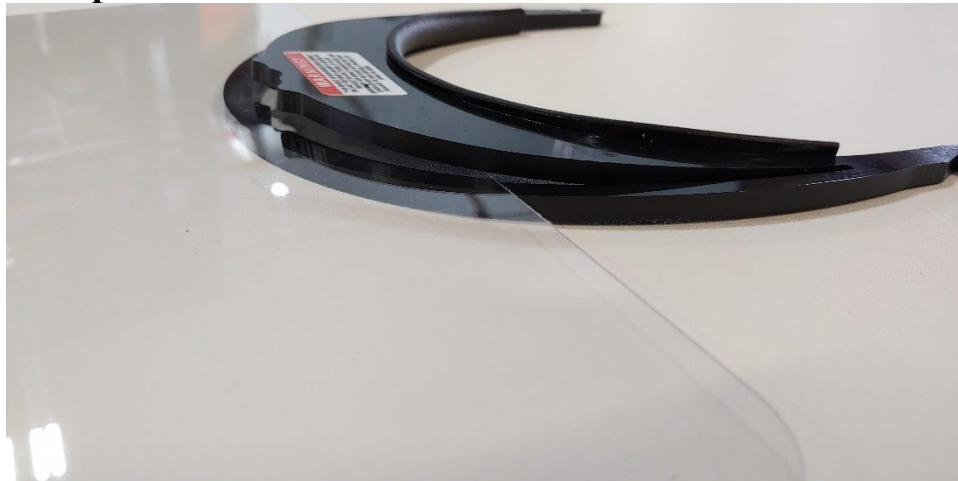


Step 3: Insert Clear Shield into the slot in the Halo while bending the Clear Shield

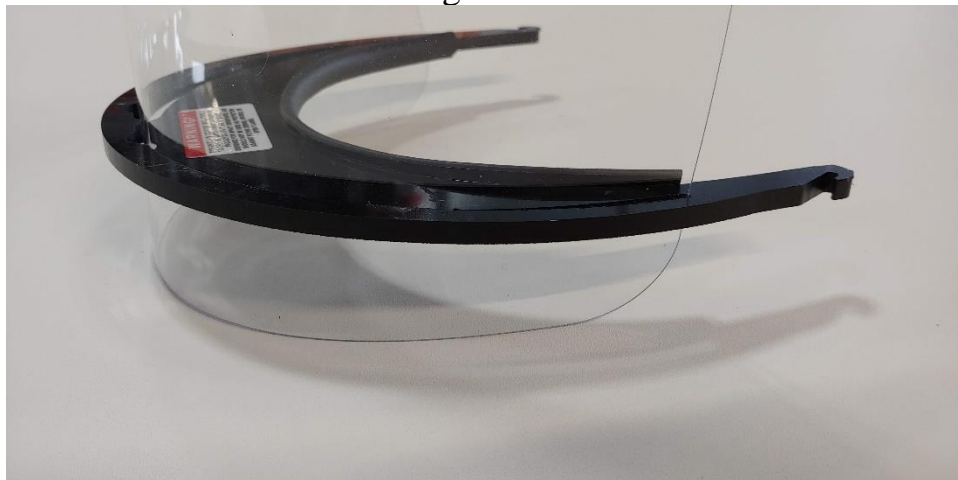

Step 4: Continue inserting **Clear Shield** into the **Halo** until the tabs on the **Clear Shield** are fully seated against the **Halo**



Step 5: Engage the tabs of the Halo into the Clear Shield

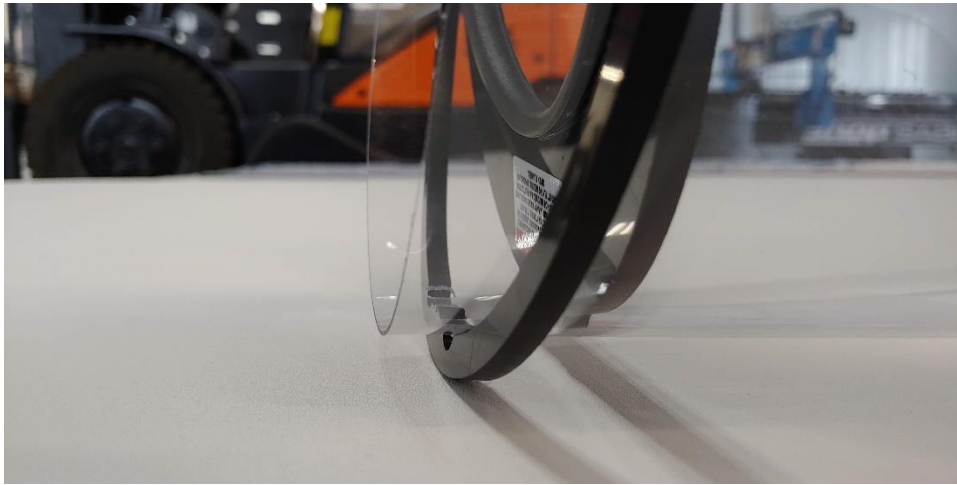

Step 6: Hook Adjustable Strap onto one side of the Halo



Step 7: Hook Adjustable Strap onto other side of the Halo



Alternative Assembly Instructions

Step 1b: Hook one **Clear Shield** tab into the slot in the **Halo**



Step 2b: Bend **Clear Shield** and insert the opposite tab into the slot on the **Halo**



Step 3b: Push inner **Halo** down to engage the tabs into the **Clear Shield**



Step 4b: Push outer **Halo** down to engage the tabs into the **Clear Shield**



Step 5b: Hook **Adjustable Strap** onto one side of the **Halo**



Step 6b: Hook **Adjustable Strap** onto other side of the **Halo**



Assembly Instruction Video

