


## InQu Matrix

1. Remove clear pouch from individual carton.
2. Peel open the clear plastic pouch.
3. Using sterile technique, transfer contents to a sterile field.
4. When ready for use, peel inner pouch thereby exposing product.
5. InQu Matrix can be cut with a scalpel or scissors and shaped by hand.
6. Autologous bone, blood, or saline may be combined with product. For use as a bone graft extender in the spine, InQu should be mixed with autograft\*.

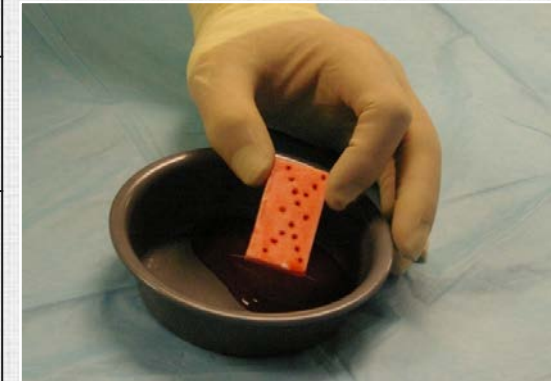

### Bone Graft Extender\*

Combined with autograft.  
Used throughout the spine.

### Bone Graft Substitute

Used alone.  
Used throughout the extremities & pelvis.

**Matrix** comes in strips of the following sizes...

**5 x 5 cm**

**5 x 10 cm**

## Additional Matrix shaping & molding tips:

- Warming to body temperature, such as by soaking packaged InQu MATRIX in warm sterile saline, will increase its flexibility
- Matrix should be shaped with the raised cut lines on the inside of the bend
- Graft may be packed into the shaped Matrix strip

