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

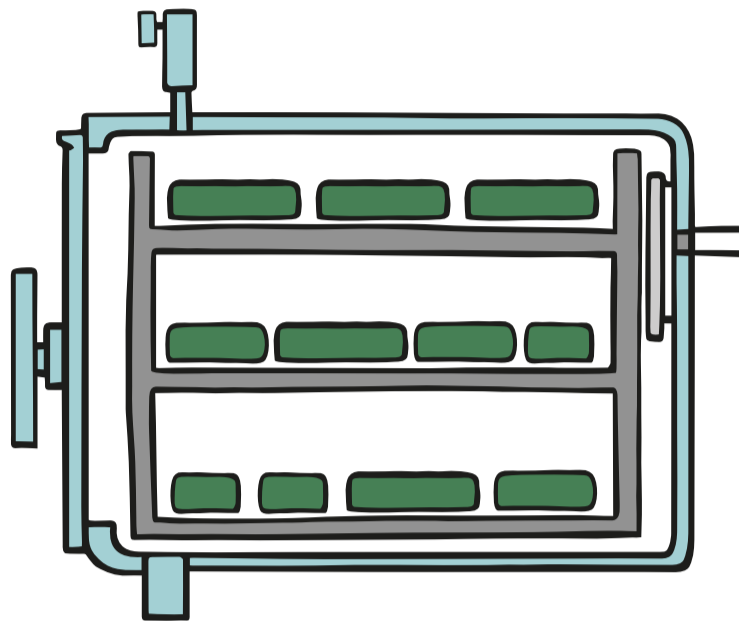
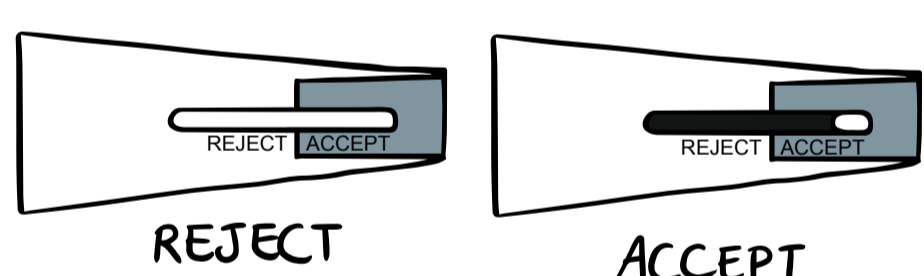
Internal indicator should be placed inside every package to determine whether the items have been exposed to adequate sterilization times and temperatures.

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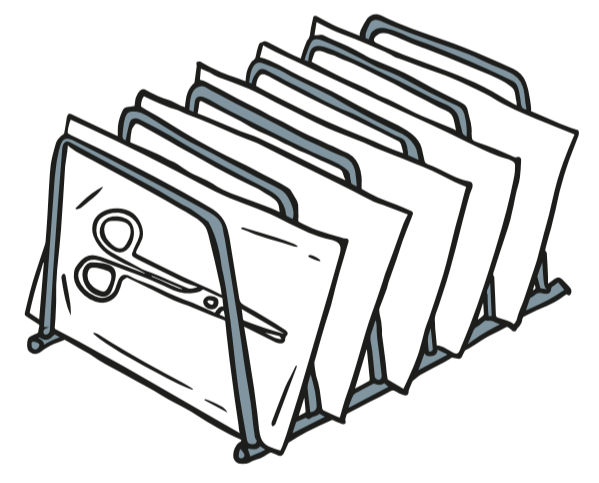Paper-plastic pouches should be placed on edge, facing paper-to-plastic.

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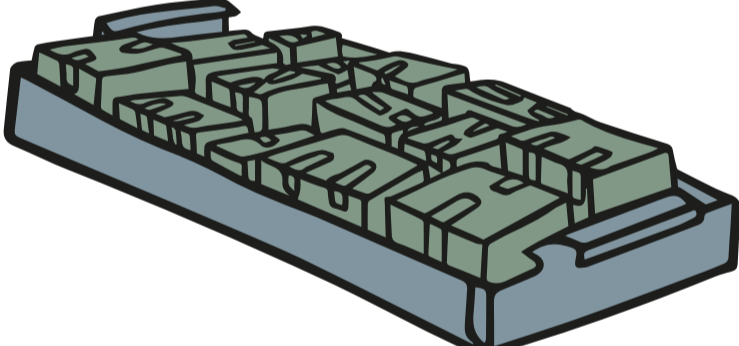There should be sufficient space between all packages to ensure efficient steam circulation.

Minimum cycle times for gravity-displacement steam sterilization cycles:

Item	Exposure time at 121°C (250°F)	Exposure time at 132°C (270°F)	Exposure time at 135°C (275°F)	Drying times
Wrapped instruments	30 minutes	15 minutes	10 minutes	15-30 minutes
Textile packs	30 minutes	25 minutes	10 minutes	15 minutes
Wrapped utensils	30 minutes	15 minutes	10 minutes	15-30 minutes
Unwrapped nonporous items (e.g. instruments)		3 minutes	3 minutes	0-1 minutes
Unwrapped nonporous and porous items in mixed load		10 minutes	10 minutes	0-1 minutes

Minimum cycle times for dynamic-air-removal steam sterilization cycles:

Item	Exposure time at 132°C (270°F)	Exposure time at 135°C (275°F)	Drying times
Wrapped instruments	4 minutes	3 minutes	20-30 minutes
Textile packs	4 minutes	3 minutes	16 minutes
Wrapped utensils	4 minutes	3 minutes	5-20 minutes
Unwrapped nonporous items (e.g. instruments)	3 minutes	3 minutes	3 minutes
Unwrapped nonporous and porous items in mixed load	3 minutes	3 minutes	20 minutes
	4 minutes	3 minutes	16 minutes
	3 minutes	3 minutes	N/A
	4 minutes	3 minutes	N/A