




video baby monitor with  
**breath detection**



quick start guide


# download the app

Your Smart Beat app will **walk you through** the setup process step-by-step.

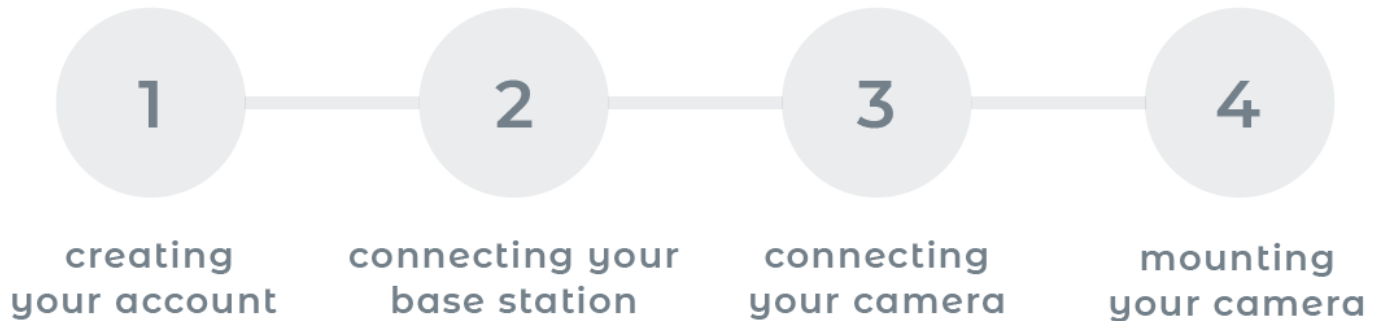


Google Play and the Google Play logo are trademarks of Google LLC.

## smart beat components

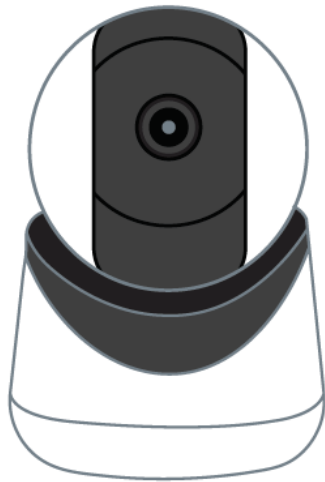


camera sends video to  
base station

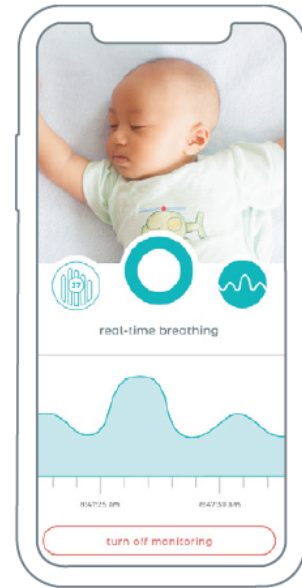

base station sends video +  
breathing status to app

## connecting base station

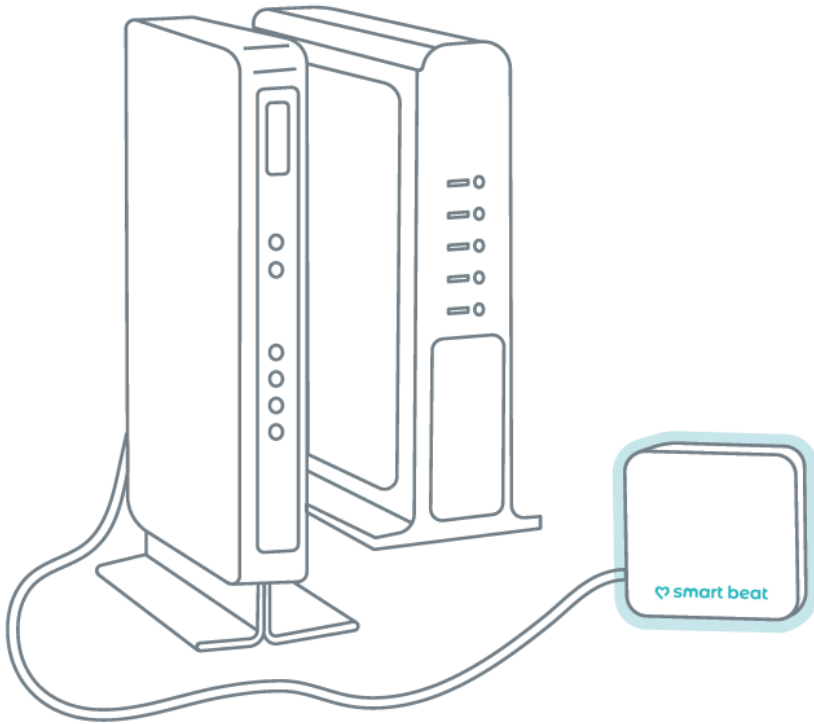


### camera sends video to base station

Tip: Be sure to connect the base station to the WiFi **router**, not the modem\*.

How to tell the difference: The internet cable coming into your house plugs into the **modem**.

## mounting your smart beat

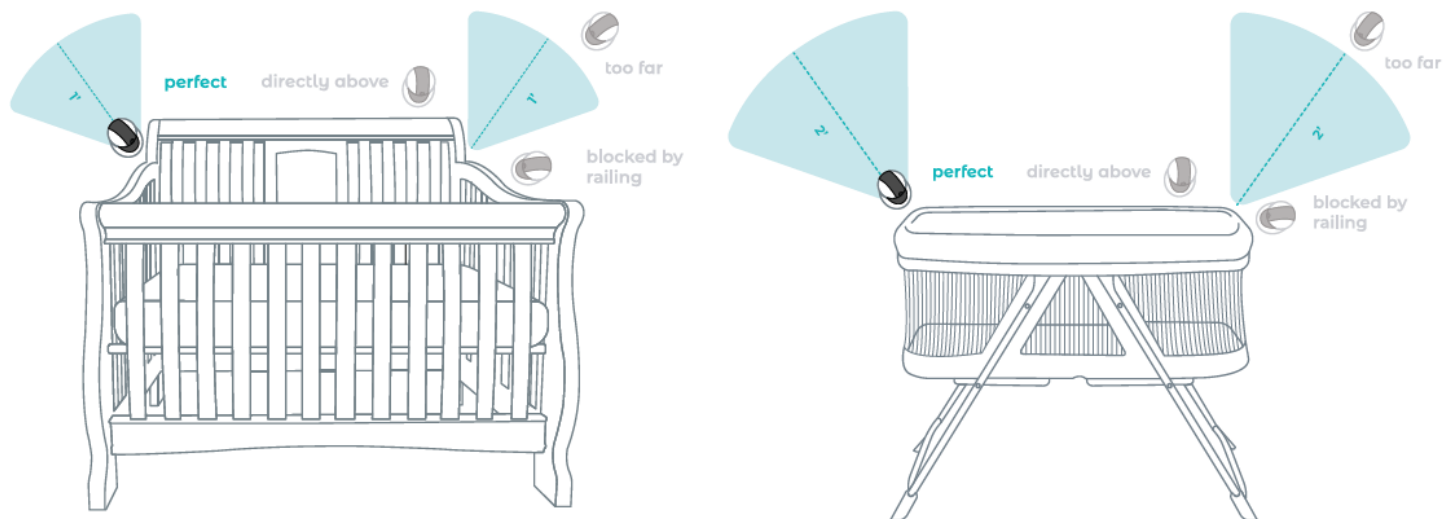

### cribs + bassinets

Tip: Mount your Smart Beat within **1 foot** of the corner of the crib, and within **2 feet** of the corner of the bassinet, far enough to be out of reach (blue zones).

## using your smart beat



click the status bar to see breathing rate and real-time breathing charts

tap the screen to access additional features, like microphone, image capture, mute, and pan/tilt.





**swipe left to activate nightstand mode** so you can see breathing status at a glance. Nightstand mode also saves battery and bandwidth.

**alerts include:** baby is asleep, baby is awake, breathing rate is high, breathing rate is low, and breathing stopped.



**two-way audio**  
Listen in and reassure without the disruption of walking into the room



**streaming video**  
HD video streams straight to your smartphone or tablet



**night vision**  
Crisp views even in the dark with infrared lights



**motion & breath detection**  
Analyzes & documents each breath



**breathing rate chart**  
Know that the baby's breathing is at healthy levels



**real-time breathing chart**  
See every individual inhale and exhale as it happens



**data encryption**  
Complete data encryption protects your privacy



**digital zoom**  
Get a closer look you can control within the app



**nightstand mode**  
Battery-saving app view you can see even without glasses on



**pan & tilt**  
See the entire room



**iOS & Android**  
Native apps for your phone & tablet



**remote viewing**  
Check in from work or date night

## important precautions

Smart Beat does not make any medical claims or guarantees that it can save a child's life in an emergency. Smart Beat is promising peace of mind by displaying data from motion detected. Here are some tips to make sure the device is working properly and to encourage the safest sleep possible.

Remove excess clothing or blankets when your child is sleeping to avoid choking hazards.

Ensure Smart Beat is not placed directly over your child's sleeping space to avoid being pulled down. Keep the camera's power cable securely attached to cable guides to avoid curious hands reaching them. Place Smart Beat in a position to have a full view of the interior of the sleeping space without crib bars or other obstructions blocking the camera view.

Make sure your child's birth date is accurate so the appropriate breathing range can be computed.

## warning!

This baby monitor uses a WiFi connection to view or hear activity. Built-in security between the camera and the cloud server improves overall camera security. To prevent unauthorized users from hearing or viewing activity, always use strong WiFi passwords.

## disclaimer

Customers use this camera and monitor at their own risk, and Smart Beat, its members, employees, agents, contractors, representatives, manufacturers, distributors and associates (Smart Beat) shall not be held liable for any loss, damage or injury suffered by the customer or any other person arising from, or in connection with, the use or manufacture of the camera and monitor, including loss, damage or injury suffered by the customer or any other person as a result of any defects in respect of the monitor, and all warranties, and any other liability from whatsoever cause arising, are hereby excluded and the customer hereby indemnifies Smart Beat in respect of any such liability. Smart Beat guarantees this product to be free from defects in materials and workmanship for a period of 12 months from the original date of purchase, verified by a sales receipt.



# help

Send an email to

[hello@mysmartbeat.com](mailto:hello@mysmartbeat.com)

or find our customer support  
live chat at

[mysmartbeat.com](https://mysmartbeat.com)

 **smart beat**

