UNIVERSAL MODULAR SPLINT

GEN 3

CLEANING AND MAINTENANCE GUIDELINES

IMPORTANT:

After multiple uses and cleaning SHEETSPLINT can degrade. The user must discard and replace the device if they identify any substantial functional defects that hinder the performance of the device. Do not use SHEETSPLINT after it has been used for training purposes. If it is not possible to clean SHEETSPLINT, then the device should be discarded after use.

CLEANING:

1. Use lukewarm water with mild detergents to soak and clean the device.
2. Use a sponge or a soft brush to remove heavy concentrations of contaminants. Do not use sharp objects to clean the device. Pay extra attention to nooks and crannies for hard-to-detect contaminants.
3. Rinse SHEETSPLINT thoroughly in lukewarm water until all detergent has been removed.
4. Inspect the device. If contaminants can not be removed, then discard and replace the device.
5. Use non-corrosive disinfectants to disinfect the device. Do not use chemicals that are harmful to polyethylene. After disinfection, allow the disinfectant agent to dry up.
6. Please note that using high concentration isopropyl alcohol to disinfect the device may degrade the printed information on the device.

MAINTENANCE:

1. Assess SHEETSPLINT's plastic surface. Other than the pre-formed ridges that are intended to fold the device in specific locations, there should not be any bends or additional folds. Inspect for any possible usage-related cracks. Deformation of the laminate sheet at folding areas is acceptable.
2. To store the unpacked device, z-fold the device to compact configuration, or store it as a flat module.

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