



STERICASE MULTI-APPLICATION INSTRUCTIONS



OZONE INSTRUCTIONS FOR A 30 MINUTE STERILIZATION CYCLE

- Carefully lift the aluminum case out of the box using the attached shoulder straps to avoid bumping the ozone intake nozzle.
- Remove the blue nozzle protector on the left side of the case
- Open the case to remove the Ozone Generator and place it next to the case.
- Remove the locking cash drawer.
- Firmly insert one end of the silicon tube into the ozone generator and the other end onto the stainless steel intake nozzle on the side of the case.
- Fill the case according to your requirements.
- Close the case with the latches pressed down.
- Plug in the ozone generator and set the TRI-X3 Ozone generator to 5 minutes. Do not exceed 5 minutes.
- You may notice a slight Ozone odor during the sanitization period. Thi is below OSHA safety standards which is tested with a precision Ozone Meter as shown below. Do not stand over the case for the first 10 minutes.

INSTRUCTIONS

- Set the hand timer to 30 minutes.
- Press start and the timer will count down to ZERO
- The timer provides an additional 25 minutes for the Ozone to dissipate
- **IMPORTANT** - Wait for the timer's loud series of beeps after 30 minutes before opening the case.
- Do not stand directly over the case during the sanitization period.

SET THE OZONE GEN.TO 5 MIN.

SET THE HAND TIMER TO 30 MIN.

OSHA COMPLIANCY VERIFIED WITH OUR OZONE TESTING EQUIPMENT



ELECTRONICS

LAPTOPS & KEYBOARDS

MEDICAL TOOLS

MEDICAL EQUIPMENT



STERICASE can sanitize anything such as tools, computer keyboards, gloves, face masks, personal items and much more.

