Naturally better

## Woodcast Solid

- EN** User guide
- DE** Benutzeranleitung
- FR** Manuel d'utilisation
- IT** Manuale d'uso
- NL** Gebruiksaanwijzing
- FI** Käyttöohje
- SE** Bruksanvisning
- NO** Bruksanvisning
- DK** Brugervejledning
- SI** Navodila za uporabnike
- HR** Korisnički vodič
- KR** 사용자 안내서

**Woodcast Solid is a completely non-toxic and perfectly mouldable and remouldable material which is particularly suited for making orthoses for occupational therapy purposes. Manufactured from clean wood and biodegradable plastic, Woodcast can be moulded without water – or rubber gloves.**

### **Purpose of use:**

Woodcast Solid is suitable for the moulding of orthoses for occupational therapy purposes. The product comes in various strengths and sizes. All Woodcast products can be combined with each other.

### **Properties:**

Woodcast is heat mouldable. At room temperature, it is as hard as wood, but once heated it becomes soft, pliable and self-adhesive. The level of support and rigidity can be adjusted by layering and combining different Woodcast products.

Cooling of the material can be accelerated by using a cold pack, for example. A moulded product can also be reheated and remoulded as necessary. Once set, the splint is lightweight, durable and x-ray transparent. If necessary, the patient's orthosis can be washed in water under +40°C, using a brush and a mild detergent such as hand soap. Because Woodcast products are non-toxic, no protective gear is required when handling them.

### **Instructions for use:**

1. Heat the Woodcast products in the Woodcast heating device. Depending on the thickness of the product, you can cut the product to the desired shape and size when it is either cold or heated. Woodcast products are ready to be moulded for a patient once they are completely soft.
2. Place both a stockinette and padding over the limb, covering the entire area under the splint.
3. **Important! If you have a patient with sensitive skin, such as a child or an elderly person, use two stockinettes instead of one and cover the entire area under the splint with padding.** Sufficient padding is important: it will protect the skin from any unnecessary and excessive heat burden.
4. **Important! Always ensure that the Woodcast product is not too hot for the patient!** If necessary, wait a moment for the material to cool down.
5. Place the product on the padding and mould it into the desired shape. If necessary, you can assist stabilisation of the limb by wrapping the cast in a thin bandage that allows the heat to pass through. Do not apply a thick layer of bandaging before the Woodcast material has cooled down – this may prevent the heat from dissipating away from the skin.
6. You can accelerate the cooling of the material by using for ex. a cold pack.
7. Finish off the cooled and solidified splint with either an appropriate bandage or Velcro fasteners.
8. If necessary, you can continue moulding the support by reheating it with the Woodcast Express heater as necessary. **In such an event, you should also ensure that the padding materials you use are heat-proof.**

### **Heating times using the Woodcast® A001 heating device:**

- Woodcast® 2 mm: 8 – 10 min
- Woodcast® 4 mm: 15 – 20 min

### **Heating times using the Woodcast® Express A002 heating device:**

- Woodcast® 2 mm: medium cycle (approx. 100 seconds)
- Woodcast® 4 mm: long cycle (approx. 3 minutes)
- Long splints: in the Woodcast® Express A002 heater, use only preformed

Woodcast splints. Use a spacer when heating folded splints. Choose the appropriate heating programme and heat the splint on both sides for the full duration of the heating programme.

### **Moulding and setting times:**

- Woodcast® 2 mm: 3 – 5 min, fully set 5 – 10 min
- Woodcast® 4 mm: approx. 5 min, fully set 10 – 15 min

### **Warnings and precautions:**

1. **Always ensure that the Woodcast product is not too hot for the patient!**
2. When moulding an orthosis, always use at least one stockinette and one layer of padding over the entire area covered by the orthosis.
3. Do not heat Woodcast products in a water bath.
4. Woodcast is self-adhesive only when heated. When combining various products, make sure that no padding or bandaging materials are left between the products, and that you press the products together with sufficient force to ensure proper adhesion.
5. Always remove the orthosis from the patient for the entire time that you reheat it. When reheating the materials, you must also ensure that any padding you use is heat-proof.
6. Make sure that the Woodcast material you are using is sufficiently strong for the indication in question.

### **Storage and disposal:**

Store Woodcast products in a dry place, protected from humidity, at a temperature lower than +40°C and out of direct sunlight. Used Woodcast orthoses can be burned as an energy source or disposed of as biowaste.

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**Onbone Oy**, Arkadiankatu 4 – 6, FI-00100 Helsinki, Finland