Naturally better

Express Heater

A002

Lämmityslaite | Heating Device | Dispositivo
di Riscaldamento | Heizgerät | Värmäre
Oppvarmingsenhet | Varmer | Verwarmingsapparaat
Dispositif de chauffage | 가열 장치 | Grelna naprava
Uređaj za zagrijavanje | Calentadores

| | | | |
|-----------|-------------------|-----------|------------------------|
| FI | Käyttöohje | NL | Handleiding |
| EN | User guide | FR | Manuel d'utilisation |
| IT | Guida utente | KO | 사용자 안내서 |
| DE | Benutzeranleitung | SI | Navodila za uporabnike |
| SE | Bruksanvisning | HR | Korisnički vodič |
| NO | Bruksanvisning | ES | Guía del usuario |
| DK | Brugervejledning | | |

Valmistaja | Manufacturer | Fabbicante | Hersteller | Tillverkare | Produsent
Producent | Fabrikant | Fabricant | 제조업체 | Proizvajalec | Proizvođač | Fabricante:

Onbone Oy, Arkadiankatu 4 – 6, FI-00100 Helsinki, Finland L135.03 – 11.12.2017
www.onbone.com

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The Onbone Express heating device is intended only for use with Onbone products. The heating cabinet is appropriate for quickly heating Onbone products as well as for re-heating preformed Onbone splints or casts. Do not use the device against instructions by manufacturer.

Before use

- Remove the protective shipping material from the heating cabinet.
- Place the device on a solid, well-aligned worktop or table, so that disconnecting of the device is easy.
- Do not place the device next to a tap or sink.
(Danger of water sprays or splashes!).
- Ensure that the main switch is OFF before plugging in the device.

Using the Onbone Express heating device

1. Switch the power ON.
2. Open the hatch, place the Onbone product on the heating area and close the hatch.
3. Choose an appropriate heating programme:
 - Short cycle (approx. 40 sec): 1 mm Ribbon
 - Medium cycle (approx. 100 sec): 2 mm and 2 mm Soft
 - Long cycle (approx. 3 min): 4 mm

Do not remove the product until the heating cycle is complete and the orange light is off.

4. At the end of the heating cycle, open the hatch and ensure that the product is soft and malleable and not too hot for the patient.
5. Apply the product according to the instructions.

Cancelling the heating cycle

- During heating cycles, the green programme light is illuminated.
- To cancel the programme, press the selected programme switch again.

Re-heating Onbone products

- When re-heating, always keep the hatch open to monitor the progress of softening.
- When re-heating, always use a shorter programme or interrupt the heating programme manually by pressing the selected programme button.
- Do not insert pre-shaped Onbone products that are larger than the hatch opening.
- Always take care that the Onbone product does not come into contact with the IR heating element itself.

Make sure that the Onbone product is not too hot for the patient!

Warnings

- Do not use the device if its' connecting cable or any other component part is defective.

- Do not replace detachable mains supply cords by inadequately rated cords.
- Risk of burns. Do not touch the Infrared heating element.
- Risk of ignition. Long splints: use only Onbone splints. Do not try to fold unheated sheets yourself.
- Risk of ignition. When heating pre-padded Onbone products, ensure that the padding material is heat resistant.

Cleaning

Unplug the device prior to cleaning. Wipe the device with a damp cloth. To clean the device use a general purpose detergent, soap or diluted ethanol (70 vol-%). Allow the device to dry prior to next use! Do not use abrasive cleaners. Remove any residues gently with a microfibre cloth. Take care not to scratch the cabinet surfaces.

Troubleshooting

Unplug the device and contact your sales person. Do not attempt to repair the device by yourself!

Warranty, Service and Disposal

The device has a 1 year warranty. Please contact the retailer in the event of any warranty issues. The device can be disposed as electrical waste in accordance with local regulations.

Technical Specifications

Power Supply 230 V AC, 50 Hz

Output power 1500 W


Weight approximately 10 kg

User interface: ON/OFF indicator light, hot surfaces warning light, programme switches.

Operating conditions: indoors, temperature range 5 – 40 °C, and at maximum of 2000 m from sea level, and maximum relative humidity 80 % (31 °C) – 50% (40 °C). Non-condensing. Power connection: Safety grounded IEC power supply connector (C14). Internal protection: replaceable 8 AT (250 Vac) (slow blow) glass tube fuse 5 x 20 mm, in both phase and neutral conductors. Programmatic resetting overheating protection (100 °C). Non-resettable thermal protection (128 °C). Safety class I. Enclosure class IP 21.

The product conforms to the following standards:

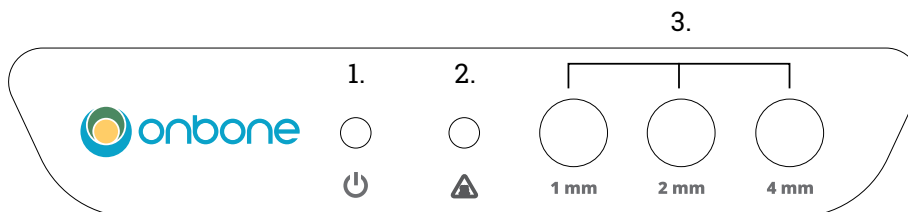EN 61010-1 and IEC 61010-2-010.

User Interface

1. On/off indicator light
2. Orange hot surfaces warning light
3. Programme selection switches

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