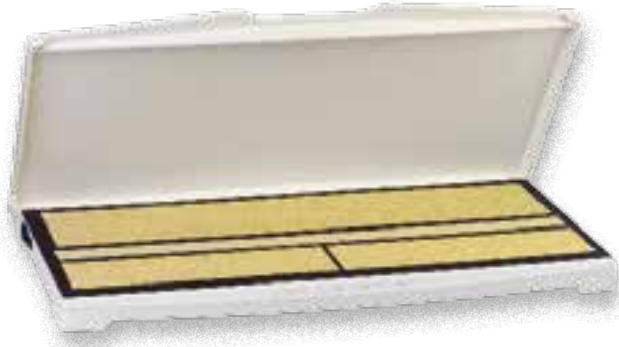
Naturally better

Flatbed Heater

A001

Lämmityslaite | Heating Device | Dispositivo di
Riscaldamento | Heizgerät | Värmäre
Oppvarmingsenhet | Varmer | Verwarmingsapparaat
Dispositif de chauffage | 가열 장 | Grelna naprava |
Uređaj za zagrijavanje | Calentadores

FI	Käyttöohje	NL	Handleiding
EN	User guide	FR	Manuel d'utilisation
IT	Guida utente	KO	사용자 안내서
DE	Benutzeranleitung	SI	Navodila za uporabnike
SE	Bruksanvisning	HR	Korisnički vodič
NO	Bruksanvisning	ES	Guía del usuario
DK	Brugervejledning		

Valmistaja | Manufacturer | Fabbricante | Hersteller | Tillverkare | Produsent |
Producent | Fabrikant | Fabricant | 제조업체 | Proizvajalec | Proizvođač | Fabricante

Onbone Oy, Arkadiankatu 4 – 6, FI-00100 Helsinki, Finland L134.02-4.11.2016
www.onbone.com

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The Onbone heating device is intended for use with Onbone products only. The device can be used to heat splint-like Onbone products. Do not use the device against instructions by manufacturer.

Before use

- Place the device on a solid, well-aligned worktop or table.
- Place the device on a solid, well-aligned worktop or table, so that disconnecting of the device is easy.
- Do not place the device next to a tap or sink. (Danger of water sprays or splashes!)

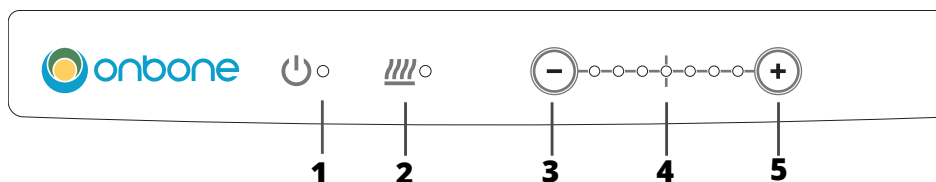
Using the Onbone A001 heating device

1. Switch the power ON.
2. Open the lid, place the Onbone product on the heating area and close the lid. The heater will switch on after you have closed the lid.
3. Heating times using the Onbone A001 heating device:
 - 1 mm Ribbon: 3 – 5 min
 - 2 mm and 2 mm Soft: 8 – 10 min
 - 4 mm: 15 – 20 min
4. Once the product is soft and malleable, it is ready to be moulded to fit the patient. Ensure the product does not feel too hot.
5. Fit the product according to the instructions

NB! The device will switch off if you open the lid. The heater will switch on again once you close the lid.

Adjusting the temperature

The default temperature setting is around 67 °C, which is the heating temperature when the middle temperature indicator is lit. Use the plus and minus buttons on the control panel to lower or raise the temperature as required.

1. On/Off indicator light
2. Heating indicator light
3. Temperature adjustment button (-)
4. Temperature indicator lights (min. 61 °C, mid-position 67 °C, max 73 °C)
5. Temperature adjustment button (+)

Make sure that the Onbone product is not too hot for the patient!

Warnings

- Do not use the device if its connecting cable or any other component part is defective.
- Do not replace detachable mains supply cords by inadequately rated cords.
- Do not keep the device switched on needlessly when it is not in use.
- Do not use this Onbone heating device to reheat splints that have already been moulded. Instead, use the Onbone Express A002 heating device to reheat splints and orthoses that have already been moulded.

Cleaning

Unplug the device prior to cleaning. Wipe the device with a damp cloth. To clean the device use a general purpose detergent, soap or diluted ethanol (70 vol-%). Do not use abrasive cleaners. Remove any residues gently with a microfibre cloth or damp cloth. Take care not to scratch the cabinet surfaces. If necessary, you can also clean the device with more conventional antiseptic wipes.

Troubleshooting

Unplug the device and contact your sales person. Do not attempt to repair the device yourself!

Warranty, Service and Disposal

The device has a 1 year warranty. Please contact the retailer in the event of any warranty issues. The device can be disposed as electrical waste in accordance with local regulations.

Technical Specifications

Power Supply 230 V AC, 50 Hz

Output power 1300 W

Weight approximately 10 kg

User interface: 2 temperature adjustment buttons (+ and -), 9 LED temperature indicators.

Operating conditions: indoors, temperature range 5-40 °C, and at maximum of 2000 m from sea level, and maximum relative humidity 80 % (31 °C) – 50% (40°C). Non-condensing.

Power connection: Safety grounded IEC power supply connector (C14). Internal protection: Replaceable 8 AT (slow blow) glass tube fuse, in both phase and neutral conductors. Programmatic resetting overheating protection (90 °C). Non-resettable thermal protection (128 °C). Safety class I. Enclosure class IP 21.

The product conforms to the following standards:

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