

# tournIQ®

The **smart** tourniquet.  
Defining best practice for IV access!

**Single-use** - Hygienic and minimizes risk for contamination

**Optimal pressure indication** - Gives larger and more palpable veins

**User-friendly** - Easy to use and comfortable for patients

**Sustainability** - Recyclable material and weighing less than 1 gram



### PRODUCT DESCRIPTION

tournIQ® differs from other tourniquets by combining the best features from all commonly used aids into **one professional tool for intravenous access**. Like most reusable tourniquets, tournIQ® is very easy to use with additional benefits such as **ensuring hygiene** through single-use and a unique indicator for **control over pressure** much like the manual blood pressure cuff.


The indicator guides users to apply **optimal pressure**, irrespective of the clinician's level of experience.<sup>1</sup> Optimal pressure means **improved vein visibility, palpability, and size**. Several studies have identified these parameters as the most important addressable factors for successful IV access.<sup>2,3,4</sup>

tournIQ® is manufactured in a very efficient production process using thin paper-like material (Tyvek®), which not only provides **strength, durability** and **patient comfort** but also enables a far more **environmentally friendly** choice compared to most alternatives.





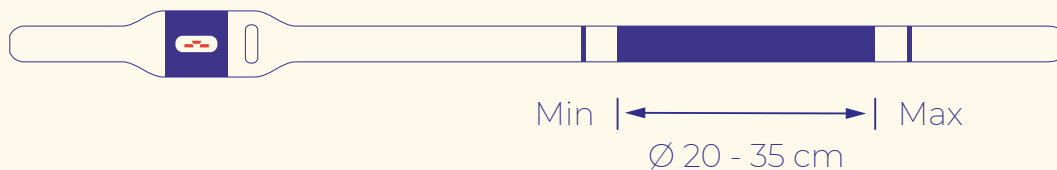


1. Ortrud product test, data on file (2017), 2. Ortrud Ultrasonic test, data on file (2016), 3. Van Loon et al. 2016 4. Mori et al. 2011.

### PRODUCT SIZE

STANDARD - Suitable for most patients. Arms of small to medium size circumference.



### USER GUIDE

<b>1</b> 	<b>2</b> 	<b>3</b> 	<b>4</b> 	<b>5</b> 	<b>6</b> 	
Loop tournIQ around the arm, and through the opening.	Arm fits in range = correct position and choice of tournIQ size.	Peel of the adhesive on the back.	Pull both ends to apply pressure - stop and hold when indicator lines are aligned.	Maintain tension and lock tournIQ by securing the adhesive end.	TournIQ is easily released with one-hand.	Instruction video

### ORDER INFORMATION

Art. No.	Description	Packaging
TOURNIQ-S50	tournIQ® - Standard. Pressure-indicating single-use tourniquet (Pocket pack).	50 pcs. / box
TOURNIQ-S100R	tournIQ® - Standard. Pressure-indicating single-use tourniquet (Small refill).	100 pcs. / refill
TOURNIQ-S250R	tournIQ® - Standard. Pressure-indicating single-use tourniquet (Large refill).	250 pcs. / refill
DISPENSER-S100	Dispenser suitable for fixing to a clinical trolley (used with TOURNIQ-S100R refill).	1 unit
DISPENSER-S250	Dispenser suitable for mounting to a wall (used with TOURNIQ-S250R refill).	1 unit