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Deutsch

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Made in Korea:
THYSOL Hitech – Ecoworks Korea
17-40, Hwaseong-ro 1424
Gyeonggi-do, Republic of Korea

User manual

English

TRIGGid Kinesiology Tape is an elastic, adhesive tape without medicines and is made of high-quality cotton with an acrylic adhesive layer. TRIGGid Kinesiology Tape can be worn for several days, is free from latex and is water resistant.

TRIGGid Kinesiology Tape may only be used by professional users who have had training in the special kinesiology-based taping technique.

Directions for use:

- In rare cases, the use of TRIGGid Kinesiology Tape can cause skin irritations or allergic reactions. The treatment should be stopped immediately if adverse reactions occur and the tape should be removed from the skin.
- It is recommended that the user first applies a small piece of tape if he/she is over-sensitive to the use of plasters or tape. It must then be borne in mind that allergic reactions do not always occur immediately, but that a later reaction (after approximately 24-48 hours) can also occur. Remove the product immediately in case of adverse reactions.
- Do not use TRIGGid Kinesiology Tape on wounds, burns and mucous membranes, nor on the mouth, nose or eyes.
- The skin should be cleaned before applying the TRIGGid Kinesiology Tape.
- When removing the TRIGGid Kinesiology Tape, the tape should be removed slowly in the direction of the hair growth. If the removal of the tape proves difficult, it is advised to use a tape removal agent.

Instructions:

- Keep TRIGGid Kinesiology Tape at room temperature (between 4-25°C) and not in direct sunlight.
- Do not put TRIGGid Kinesiology Tape on or near radiators or any other source of direct heat.
- Keep the tape out of reach of children.
- For single use only.
- Injuries can always occur, even when TRIGGid Kinesiology Tape is used in the proper manner.

Winter, J. / Mayer / Hehner / Winter, T. GbR are not liable for any injuries and/or damage due to TRIGGid Kinesiology Tape or the unprofessional use of the product.

Winter, J. / Mayer / Hehner / Winter, T. GbR
Otto-Hahn-Straße 56
63456 Hanau
Germany
www.triggid.de

Made in Korea:
THYSOL Hitech – Ecoworks Korea
17-40, Hwaseong-ro 1424
Gyeonggi-do, Republic of Korea