



Made for DSI Ltd
59 haAvoda str.
Ashdod, 7706300 Israel
www.dsisrael.com
+972-8931-7235

Tissue Conditioner

Soft Liner



Self-curing Resin for Denture Reline

DIRECTIONS FOR USE

INDICATIONS:

As soft temporary denture reline material and tissue conditioner. As functional impression material.

1. Clean denture thoroughly by brushing with liquid detergent and water.
2. Inspect denture for proper occlusion.
3. Relieve all undercut and obvious pressure areas.
4. Add 5g of powder into the mixing cup provided. Using syringe, place 3ml of liquid (one full syringe, depending on the denture size the dosage may be doubled). Mix gently with a stainless steel spatula for 30 seconds.
5. After allowing the mixture to form a viscous, yet flowable state, pour the mixture into the denture, using a spatula to flow the material to the peripheral border to be certain that there is ample material to form an adequate peripheral (muscle-trimming) roll.
6. Rinse the mouth thoroughly with plain cold water.
7. Place the denture in the mouth and close the jaw slowly to a normal biting position to bring the teeth into proper occlusion. Hold this position for 1-2 minutes before muscle trimming.
8. After a further 3 minutes, remove the denture from the mouth and check for full coverage. Additional material can be added as required, as the material will blend homogeneously. If additional material is used, reinsert the denture for 2-3 minutes and re-inspect.



- Trim excess with warm water and a sharp knife.
- Visit dentist periodically for examination of progress during the resorption period. Additional material can be added as required after grinding and relieving areas where the base material has shown through.
- Tissue Conditioner can be readily removed from a denture by scraping with a blunt knife or spatula and then wiping the denture with alcohol before washing the denture with a mild detergent and warm water.

CLEANING:

Before sending dentures to a dental lab, always clean and disinfect following the instructions on the packet.

WARNING:

Powder and liquid may cause slight irritation to eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. The powder contains polymerised methacrylate and therefore may cause sensitisation by contact in susceptible persons. If sensitisation occurs, discontinue use.

STORAGE:

Store materials in a dry dark place. Always replace all caps immediately after use.

SHELF LIFE:

4 years from the date of manufacture when supplied in standard packaging.

SPECIALLY FORMULATED FOR USE IN DENTISTRY.

PACKAGING:

| | |
|--------------------|-------|
| Powder | 30 g |
| Liquid | 20 ml |
| Mixing accessories | |

REF: DIY-SL



FLAMMABLE

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.

